



Applicable standard: EN 14683:2019+AC:2019 YY0469-2011
Production license No.: Hunan Food&Drug Administration permission 20200023
Registration No.: Hunan medical device registration permission 20202140288
Applicable scenario: Provide protection for operating rooms doctors.
Main components: PP non-woven fabric, meltblown filter cloth, mask strap, and nose clip.
Storage conditions: It should be stored in an environment with relative humidity below 85%, no corrosive gas, and good ventilation, avoiding the high temperature.
Transportation conditions: Avoid heavy load, direct sunlight and rain and snow during transportation.



Hunan EEXI Technology & Service Co., Ltd.
Add: No.6, North of pingtou road, Liuyang Hi-tech Industrial Development Zone, Hunan, China
Hotline: +86-4000-333-088

EC REP

Shanghai International Holding Corp. GmbH (Europe)
Add: Eiffestrasse 80, 20537 Hamburg Germany



Product size: 17.5 cm x 9.5 cm
Date of production : See code on package.
Shelf life: 2 years
Quantity per box: 50 pcs
Article No.: YX011

SURGICAL TYPE IIR FACE MASK

NON-STERILE



Instructions for use

Lay the mask flat, nose clip upward; unfold it; cover it on your face and nose; put the earloop on until it perfectly fit your nose and face.

Intended use

The surgical face mask is single-use, disposable device, provided non-sterile, and intended to be used to protect both the patient and the healthcare personnel from transfer of microorganisms, body fluids and particulate material. These face masks are intended for use in infection control practices to reduce the potential exposure to blood and body fluids.

Precautions

1. Check whether the package is intact before use, confirm the marks and date of manufacture on the external package, and use it before the expiry date.
2. People allergic to PP non-woven fabrics should use it with caution.
3. After using the product, please dispose of it according to the requirements of the environmental protection agency or related authorities.
4. This product is a disposable mask. It is not recommended to clean or reuse it. If you feel uncomfortable during use, please stop using it immediately or replace it with a new one.

